

Durata™

## Defibrillation Lead

## Product Highlights

- The DF4 connector is designed to simplify implants by streamlining defibrillation connections into a single terminal pin and reducing the number of set screws
- Redundant conductors serve as a backup system in the unlikely event of a conductor failure
- Symmetrically aligned cables within the lead body and centrally located coil provide for additional protection to the inner coil
- Optim™ lead insulation – a chemical co-polymer that blends the best features of polyurethane and silicone for improved handling and increased durability
- Two innovative designs are intended to help prevent tissue ingrowth – flat-wire technology provides a low profile for the defibrillation coils, and silicone backfilling completely fills the shock coil space



## Ordering Information

Contents: Defibrillation lead

Model Number	Insulation	Fixation	Min. Introducer (F)	Shock Configuration	Sensing	Tip-to-Proximal Coil (cm)	Connector	Lengths (cm)
7120	Optim	Ext/Ret Helix	7	Dual-coil	True bipolar	17	DF1; IS-1	60; 65
7120Q	Optim	Ext/Ret Helix	7	Dual-coil	True bipolar	17	DF4	52; 58; 65
7121	Optim	Ext/Ret Helix	7	Dual-coil	True bipolar	21	DF1; IS-1	60; 65; 75
7121Q	Optim	Ext/Ret Helix	7	Dual-coil	True bipolar	21	DF4	52; 58; 65
7122	Optim	Ext/Ret Helix	7	Single-coil	True bipolar	N/A	DF1; IS-1	60; 65; 75
7122Q	Optim	Ext/Ret Helix	7	Single-coil	True bipolar	N/A	DF4	52; 58; 65
7170	Optim	Tines	7	Dual-coil	True bipolar	17	DF1; IS-1	60; 65; 75
7170Q	Optim	Tines	7	Dual-coil	True bipolar	17	DF4	52; 58; 65
7171	Optim	Tines	7	Dual-coil	True bipolar	21	DF1; IS-1	60; 65; 75
7171Q	Optim	Tines	7	Dual-coil	True bipolar	21	DF4	52; 58; 65
7172Q	Optim	Tines	7	Single-coil	True bipolar	N/A	DF4	52; 58; 65

**Indications for Use:** The Durata™ Models 7120, 7120Q, 7121, 7121Q, 7122, 7122Q, 7170, 7170Q, 7171, 7171Q and 7172Q transvenous leads are indicated for use with compatible pulse generators (refer to the applicable defibrillator manual for system indications). They provide pacing and sensing and deliver cardioversion/defibrillation therapy to the heart. A transvenous lead system may offer the patient the benefit of avoiding a thoracotomy for lead implantation. If the initial lead configuration is not effective, repositioning of the lead or other lead configurations should be attempted. In some patients, a nonthoracotomy lead configuration may not provide reliable conversion of arrhythmias, and the use of subcutaneous or epicardial patch defibrillation leads should be considered.

**Contraindications:** Contraindications for use of the Durata leads with an implantable pulse generator include ventricular tachyarrhythmias resulting from transient or reversible factors such as drug toxicity, electrolyte imbalance, or acute myocardial infarction. Transvenous lead systems are contraindicated for patients with tricuspid valvular disease or a mechanical heart valve. Durata leads are contraindicated for patients for whom a single dose of 1.0 mg of dexamethasone sodium phosphate is contraindicated. The Durata 7120, 7120Q, 7121, 7121Q, 7122, 7122Q, 7170, 7170Q, 7171, 7171Q and 7172Q leads are contraindicated for extra firm (red color knob) stylets. The lead is not designed, sold, or intended for use other than as indicated.

1. St. Jude Medical DF1 lead connectors conform to the international connector standard ISO 11318/Amd.
2. St. Jude Medical IS-1 lead connectors conform to the international connector standard ISO 5841.
3. St. Jude Medical DF4 lead connectors conform to the international connector standard ISO 27186: 2010 (E).

**Potential Complications:** Possible complications of the use of transvenous lead systems include, but are not limited to, supraventricular or ventricular arrhythmias, conduction disturbances, cardiac perforation, cardiac tamponade, loss of contractility, air embolism, heart wall rupture, myocarditis, post-operative heart failure, chronic mechanical stimulation of the heart, tricuspid valve dysfunction, lead fracture necessitating surgical removal, pneumothorax, hemothorax, infection, tissue necrosis and erosion of the skin. Specific events and effects are summarised below:

**WARNING:** Implanted cardiac leads are subjected to a hostile environment within the body due to constant, complex flexural and torsional forces, interactions with leads and/or the pulse generator, or other forces associated with cardiac contractions and patient physical activity, posture and anatomical influences. Cardiac leads' functional lifetimes can be affected by these and other factors.

Refer to the defibrillator manual for additional complications and precautions specific to the pulse generator.

**Customer Support:** 46-8-474-4756

**Brief Summary:** Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use. Devices depicted may not be available in all countries. Check with your St. Jude Medical representative for product availability in your country. Unless otherwise noted, ™ indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL, the nine-squares symbol and MORE CONTROL. LESS RISK. are registered and unregistered trademarks and service marks of St. Jude Medical, Inc. and its related companies. ©2011 St. Jude Medical, Inc. All rights reserved.



**ST. JUDE MEDICAL**  
MORE CONTROL. LESS RISK.

Durata™

## Defibrillation Lead

## Product Specifications

## PHYSICAL SPECIFICATIONS

## True Bipolar, Active-Fixation Defibrillation Leads

Models	7120	7120Q	7121	7121Q	7122	7122Q
Fixation	Ext/Ret Helix	Ext/Ret Helix	Ext/Ret Helix	Ext/Ret Helix	Ext/Ret Helix	Ext/Ret Helix
Shock Configuration	Dual-Coil	Dual-Coil	Dual-Coil	Dual-Coil	Single-Coil	Single-Coil
Sensing Configuration	True Bipolar	True Bipolar	True Bipolar	True Bipolar	True Bipolar	True Bipolar
Min. Size Introducer	7 F	7 F	7 F	7 F	7 F	7 F
Lengths (cm)	60; 65	52; 58; 65	60; 65; 75	52; 58; 65	60; 65; 75	52; 58; 65
Connector	DF1; IS-1	DF4	DF1; IS-1	DF4	DF1; IS-1	DF4
Body Diameter	6,8 F	6,8 F	6,8 F	6,8 F	6,8 F	6,8 F
Tip-to-Anode Spacing	11 mm	11 mm	11 mm	11 mm	11 mm	11 mm
Tip-to-Proximal Coil	17 cm	17 cm	21 cm	21 cm	N/A	N/A
Tip Electrode Area	6 mm <sup>2</sup>	6 mm <sup>2</sup>	6 mm <sup>2</sup>	6 mm <sup>2</sup>	6 mm <sup>2</sup>	6 mm <sup>2</sup>
Steroid Plug	Yes	Yes	Yes	Yes	Yes	Yes
Distal Shock Coil Area	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>
Proximal Shock Coil Area	588 mm <sup>2</sup>	588 mm <sup>2</sup>	588 mm <sup>2</sup>	588 mm <sup>2</sup>	N/A	N/A

## True Bipolar, Passive-Fixation Defibrillation Leads

Models	7170	7170Q	7171	7171Q	7172Q
Fixation	Tines	Tines	Tines	Tines	Tines
Shock Configuration	Dual-Coil	Dual-Coil	Dual-Coil	Dual-Coil	Single-Coil
Sensing Configuration	True Bipolar	True Bipolar	True Bipolar	True Bipolar	True Bipolar
Min. Size Introducer	7 F	7 F	7 F	7 F	7 F
Lengths (cm)	60; 65; 75	52; 58; 65	60; 65; 75	52; 58; 65	52; 58; 65
Connector	DF1; IS-1	DF4	DF1; IS-1	DF4	DF4
Body Diameter	6,8 F	6,8 F	6,8 F	6,8 F	6,8 F
Tip-to-Anode Spacing	11 mm	11 mm	11 mm	11 mm	11 mm
Tip-to-Proximal Coil	17 cm	17 cm	21 cm	21 cm	N/A
Tip Electrode Area	3.5 mm <sup>2</sup>	3.5 mm <sup>2</sup>	3.5 mm <sup>2</sup>	3.5 mm <sup>2</sup>	3.5 mm <sup>2</sup>
Steroid Plug	Yes	Yes	Yes	Yes	Yes
Distal Shock Coil Area	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>	367 mm <sup>2</sup>
Proximal Shock Coil Area	588 mm <sup>2</sup>	588 mm <sup>2</sup>	588 mm <sup>2</sup>	588 mm <sup>2</sup>	N/A

Customer Support: 46-8-474-4756

**Brief Summary:** Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use. Devices depicted may not be available in all countries. Check with your St. Jude Medical representative for product availability in your country. Unless otherwise noted, ™ indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL, the nine-squares symbol and MORE CONTROL. LESS RISK. are registered and unregistered trademarks and service marks of St. Jude Medical, Inc. and its related companies. ©2011 St. Jude Medical, Inc. All rights reserved.

Item No. GMC783EN



**ST. JUDE MEDICAL**  
MORE CONTROL. LESS RISK.